


The Hi-Lo Remote

January 30, 2015 to present

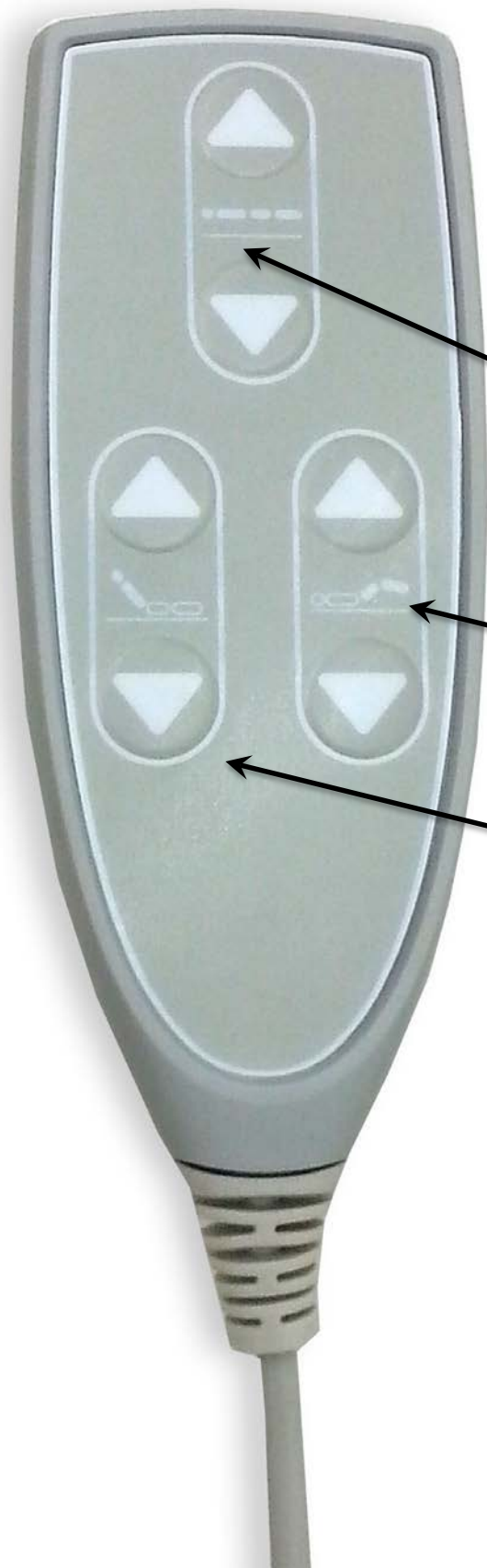

REMOTE LOCK OUT PROCEDURE:

PRESS and HOLD BOTH DOWN BUTTONS on HEAD and FOOT articulation functions for about 5 seconds. The light on the remote will flash when it is locked. To UNLOCK repeat this procedure.

HiLo Bed Surface

(Up/Down)

Knee & Foot

(Up/Down)

Head

(Up/Down)

RESET PROCEDURE:

1. Press and hold the DOWN button on the HiLo function, after the bed is down completely, continue to hold for 5-10 seconds. You will hear the control box click.
2. Your bed is now reset and all functions operate normally.